

Manage postpartum hemorrhage simply and effectively.



Bakri[™]
POSTPARTUM BALLOON



www.cookmedical.com


A Simple Solution for Postpartum Hemorrhage



Balloon tamponade is an effective option for the treatment of severe postpartum hemorrhage that occurs as a result of uterine atony, especially when medical therapy fails.^{1,2}

1. American College of Obstetricians and Gynecologists. ACOG practice bulletin: clinical management guidelines for obstetrician-gynecologists number 76, October 2006: postpartum hemorrhage. *Obstet Gynecol.* 2006;108(4):1039-1047.
2. Dabelea V, Schultze PM, McDuffie RS Jr. Intrauterine balloon tamponade in the management of postpartum hemorrhage. *Am J Perinatol.* 2007;24(6):359-64.

Treatment Modalities

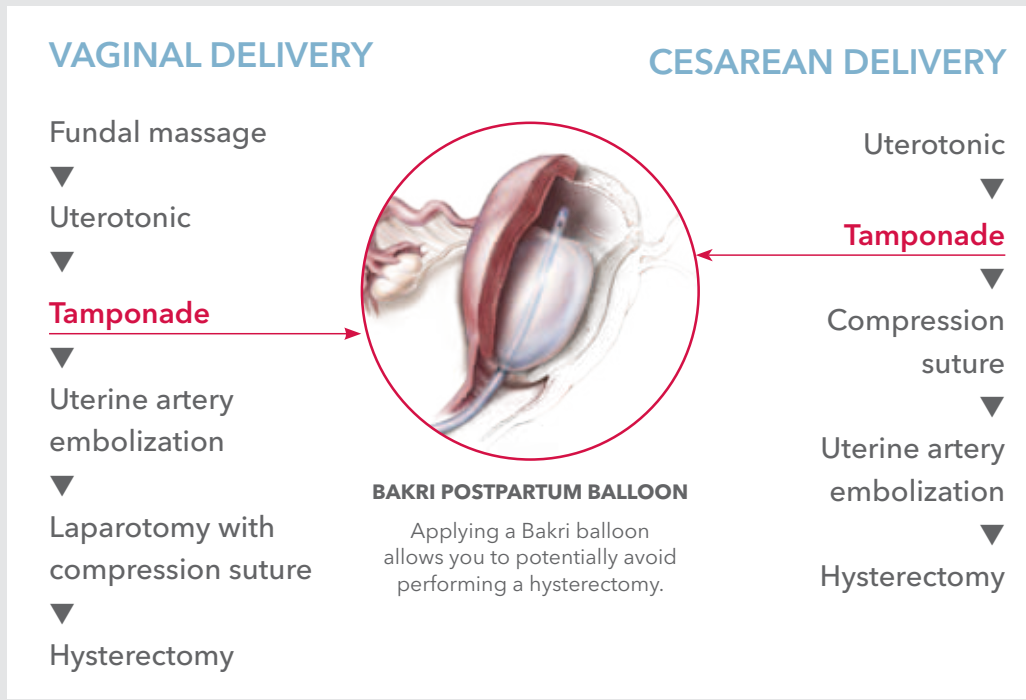

When inflated with sterile liquid, the Bakri Postpartum Balloon conforms to the shape of the uterus and may provide immediate management of postpartum hemorrhage.

- The silicone construction eliminates the problem of patient sensitivity to latex
- The balloon is easy to position and remove
- The balloon controls bleeding simply
- The drainage lumen allows blood loss monitoring
- The balloon enables conservative management of postpartum hemorrhage



Bakri™

POSTPARTUM BALLOON

Used for temporary control or reduction of postpartum hemorrhage when conservative management of uterine bleeding is warranted. Includes a 60 mL syringe. Please refer to the instructions for use for complete information on product usage and proper indications and contraindications. The Bakri balloon is supplied sterile in peel-open package and is intended for one-time use.

Order Number	Reference Part Number	Fr	Length cm	Balloon Volume mL
G30673	J-SOS-100500	24	54	500

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Resources

B-Lynch C, Keith L, Lalonde A, et al, eds. *A Textbook of Postpartum Hemorrhage*. Duncow, Kirkmahoe, Dumfriesshire, UK: Sapiens Publishing; 2006.

Bakri YN. Balloon device for control of obstetrical bleeding. *Eur J Obstet Gynecol Reprod Biol*. 1999;86:S84.

Bakri YN, Amri A, Abdul Jabbar F. Tamponade-balloon for obstetrical bleeding. *Int J Gynaecol Obstet*. 2001;74(2):139-142.

Condous GS, Arulkumaran S, Symonds I, et al. The "tamponade test" in the management of massive postpartum hemorrhage. *Obstet Gynecol*. 2003;101(4):767-772.

Nelson WL, O'Brien JM. The uterine sandwich for persistent atony: combining the B-Lynch compression suture and an intrauterine Bakri balloon. *Am J Obstet Gynecol*. 2007;196(5):e9-10.

Cover illustration by Lisa Clark.

Customer Service

EMEA: EDI - www.cookmedical.com/edi.do
 Distributors: +353 61239240, ssc.distributors@cookmedical.com
 Austria: +43 179567121, oe.orders@cookmedical.com
 Belgium: +32 27001633, be.orders@cookmedical.com
 Denmark: +45 38487607, da.orders@cookmedical.com
 Finland: +358 972519996, fi.orders@cookmedical.com
 France: +33 171230269, fr.orders@cookmedical.com
 Germany: +49 6950072804, de.orders@cookmedical.com
 Hungary: +36 17779199, hu.orders@cookmedical.com
 Ireland: +353 61239252, ie.orders@cookmedical.com
 Italy: +39 0269682853, it.orders@cookmedical.com
 Netherlands: +31 202013367, nl.orders@cookmedical.com
 Norway: +47 23162968, no.orders@cookmedical.com
 Spain: +34 912702691, es.orders@cookmedical.com
 Sweden: +46 858769468, se.orders@cookmedical.com
 Switzerland - French: +41 448009609, fr.orders@cookmedical.com
 Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
 Switzerland - German: +41 448009609, de.orders@cookmedical.com
 United Kingdom: +44 2073654183, uk.orders@cookmedical.com

www.cookmedical.com

Americas: EDI - www.cookmedical.com/edi.do
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
 E-mail: orders@cookmedical.com

Australia:
 Phone: +61 738411188, 1800777222, Fax: +61 738411288, 1800077283
 E-mail: cau.custserv@cookmedical.com

