IFU0028-4

**EchoBrush™ - Endoscopic Ultrasound  
Cytology Brush**

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## **English**

### **EchoBrush™ - Endoscopic Ultrasound Cytology Brush**

#### **INTENDED USE**

The EchoBrush™ is used to sample targeted submucosal gastrointestinal lesions through the accessory channel of an ultrasound endoscope.

#### **NOTES**

Do not use this device for any purpose other than the stated intended use. Inventory rotation of sterile products is essential. Verify the expiration date of the package label prior to using the product. If the expiration date has lapsed, do not use or resterilize the device. If the product package is open or damaged when received, do not use this device. Cook Endoscopy devices must be stored in a dry location, away from temperature extremes.

#### **CONTRAINDICATIONS**

Contraindications include those specific to the primary endoscopic procedure to be performed in gaining access to the desired injection site. Relative contraindications to cytology include, but are not limited to: coagulopathy.

#### **POTENTIAL COMPLICATIONS**

Potential complications associated with gastrointestinal endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

#### **CAUTIONS**

A thorough understanding of the technical principles, clinical applications and risks associated with endoscopic ultrasound and fine needle aspiration is necessary. The EchoBrush™ should only be used by, or under the supervision of, physicians thoroughly trained in endosonography, aspiration biopsy and brush cytology.

#### **PRECAUTIONS**

The EchoBrush™ is designed for use with the Cook Endoscopy 19 gauge EchoTip Ultra® Needle (ECHO-19).

## **INSTRUCTIONS FOR USE**

1. Upon removal of the EchoBrush™ from the packing, visually inspect with particular attention to kinks, bends, and breaks. If an abnormality is detected that would prohibit proper working condition, do not use. Please notify Cook Ireland for return authorization.
2. The EchoBrush™ may be used either before or after fine needle aspiration.
3. Remove the stylet of the endoscopic ultrasound needle and introduce the EchoBrush™ into the aspiration lumen.
4. Advance the device, in small increments, until the brush head is seen on the EUS image. As a guide, there is a white mark on the proximal end of the brush shaft near the handle of the EchoBrush™. When the white mark enters the lumen of the endoscopic ultrasound needle the brush-head will start to exit the needle-tip of the endoscopic ultrasound needle.
5. Gently brush the walls of the target lesion with the EchoBrush™ under EUS guidance.
6. It may be necessary to brush the area several times to obtain adequate cellular material.
7. To remove the EchoBrush™, gently retract it into the tip of the endoscopic ultrasound needle so that the brush is completely covered by the needle. Do not withdraw the EchoBrush™ through the aspiration lumen of the endoscopic ultrasound needle.
8. Retract the needle-tip into the sheath of the endoscopic ultrasound needle.
9. Withdraw the endoscopic ultrasound needle from the endoscope.
10. Carefully advance the needle-tip from the sheath.
11. Advance the EchoBrush™ so that it exits the needle-tip of the endoscopic ultrasound needle.

12. The brush-head of the EchoBrush™ may be removed with a wire cutter and prepared for examination according to institutional guidelines.

13. The shaft of the EchoBrush™ may then be withdrawn completely through the aspiration lumen of the endoscopic ultrasound needle.

14. Upon completion of the procedure, dispose of the device according to institutional guidelines for biohazardous medical waste.

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Wilson-Cook and EchoTip Ultra are registered trademarks of
Cook is a registered trademark of Cook Incorporated
Wilson-Cook Medical Inc.
**Cook Ireland Ltd.**