



# FluoroSet™

RADIOGRAPHIC TUBAL ASSESSMENT SET

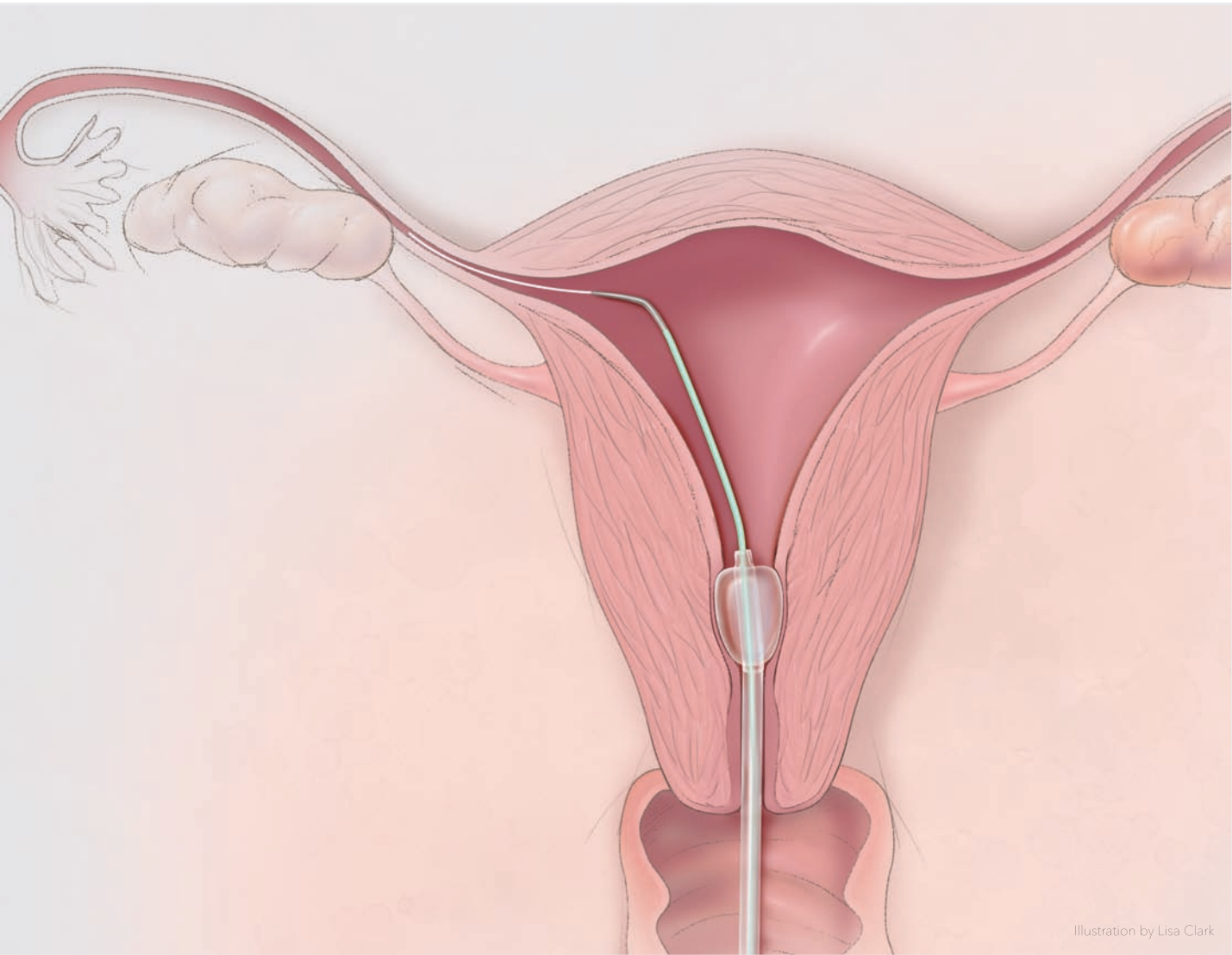


Illustration by Lisa Clark

Assess tubal patency  
quickly and easily.



# FluoroSet™

RADIOGRAPHIC TUBAL ASSESSMENT SET

## All-inclusive FluoroSet offers convenience.

The FluoroSet combines all the devices you need to introduce contrast media for fluoroscopic evaluation of both the uterine cavity and fallopian tubes, permitting quick, easy and affordable diagnosis of tubal occlusion.

### The set includes:

- Intrauterine Access Balloon Catheter
- Selective Salpingography Catheter with Beacon® Tip
- Roadrunner® PC Wire Guide with Slip-Coat® Hydrophilic Coating

Cook Medical—Simplifying procedures for faster results.


1

2


3

*Slip-Coat is a registered trademark of Angiotech Biocoatings Corp.*


## 3-step process to determine tubal patency.

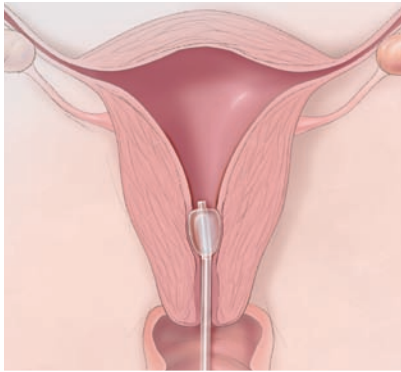

Advance the Intrauterine Access Balloon Catheter transcervically until the balloon is secure in the lower area of the uterus or cervix. Inflate the balloon with 2 mL of sterile saline. Deploy 5-10 mL of appropriate contrast media into the uterus to administer a standard hysterothy.

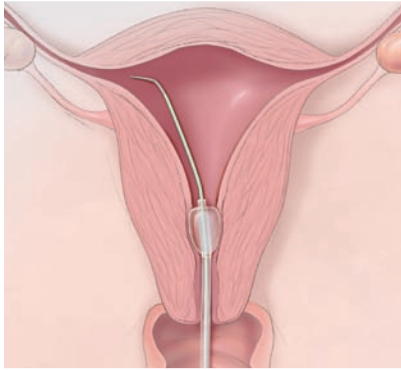

Insert the Selective Salpingography (SSG) Catheter through the working channel of the Intrauterine Access Balloon Catheter. Under fluoroscopy, rotate the catheter to access the correct fallopian tube. The distal tip is radiopaque. Appropriate contrast media may be injected through the SSG catheter in order to assess tubal patency.

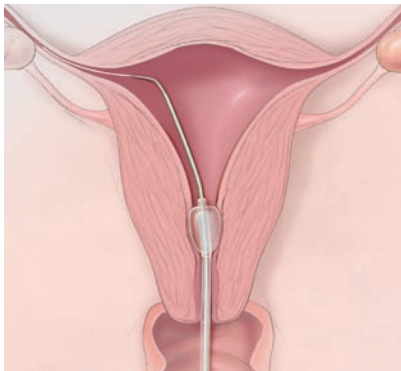


If entry to the tubes is hindered, the Roadrunner PC Wire Guide can be utilized to facilitate tubal access.

Please refer to the Instructions for Use for full procedural details and contraindications.

**COOK**  
MEDICAL

# FluoroSet™

RADIOGRAPHIC TUBAL ASSESSMENT SET

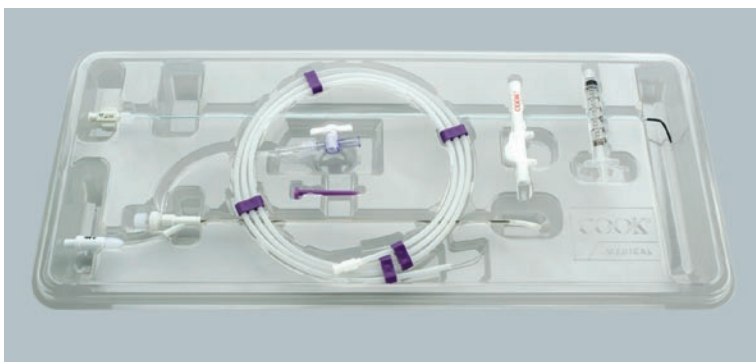

Determine tubal patency with the convenient FluoroSet—all the devices you need to introduce contrast media for fluoroscopic evaluation.

Global Product Number	Order Number	Access Catheter Fr	Injection Catheter Fr	Wire Guide Diameter inch/mm
G51608	J-RTAS-100	9.0	5.0	.035 / .89

## INCLUDED PRODUCTS

### Intrauterine Access Balloon Catheter

Fr	Length cm	Balloon Volume mL
9.0	23	2

### Selective Salpingography Catheter with Beacon Tip

Fr	Length cm	Tip Angle Degrees
5.0	40	65

### Roadrunner PC Wire Guide with Slip-Coat Hydrophilic Coating

Diameter inch/mm	Length cm	Taper Length cm
.035/.89	60	7

For more information about the FluoroSet, contact your Cook Medical representative.



[www.cookmedical.com](http://www.cookmedical.com)

**COOK MEDICAL INCORPORATED**  
P.O. Box 4195, Bloomington, IN 47402-4195 USA  
Phone: 812.339.2235, Toll Free: 800.457.4500, Toll Free Fax: 800.554.8335

**COOK IRELAND LTD.**  
O'Halloran Road, National Technology Park,  
Limerick, IRELAND, Phone: 353 61 334440, Fax: 353 61 334441

**COOK (CANADA) INC.**  
111 Sandiford Drive, Stouffville, Ontario, L4A 7X5 CANADA  
Phone: 905.640.7110, Toll Free: 800.668.0300, Fax: 905.640.6804

**WILLIAM A. COOK AUSTRALIA PTY. LTD.**  
95 Brandl Street, Brisbane Technology Park, Eight Mile Plains  
Brisbane, QLD 4113 AUSTRALIA, Phone: +61 7 3841 1188

© COOK 2009 WH-BM-FSLB-EN-200904