- EN**
3 **Sholkoff Balloon Hysterosalpingography Catheter**
Instructions for Use
- CS**
4 **Balónkový katetr Sholkoff pro hysterosalpingografii**
Návod k použití
- DA**
5 **Sholkoff ballonkateter til hysterosalpingografi**
Brugsanvisning
- DE**
6 **Sholkoff Hysterosalpingographie-Katheter mit Ballon**
Gebrauchsanweisung
- EL**
8 **Καθετήρας υστεροσαλπιγγογραφίας με μπαλόني Sholkoff**
Οδηγίες χρήσης
- ES**
9 **Catéter balón de histerosalpingografía Sholkoff**
Instrucciones de uso
- FR**
10 **Sonde de Sholkoff à ballonnet pour hystérosalpingographie**
Mode d'emploi
- HU**
11 **Sholkoff hysterosalpingográfiás ballonkatéter**
Használati utasítás
- IT**
13 **Catetere a palloncino per isterosalpingografia Sholkoff**
Istruzioni per l'uso
- NL**
14 **Sholkoff hysterosalpingografie ballonkatheter**
Gebruiksaanwijzing
- NO**
15 **Sholkoff ballongkateter for hysterosalpingografi**
Bruksanvisning
- PL**
16 **Cewnik balonowy Sholkoff do histerosalpingografii**
Instrukcja użycia
- PT**
18 **Cateter de balão Sholkoff para histerossalpingografia**
Instruções de utilização
- SV**
19 **Sholkoff hysterosalpingografikateter med ballong**
Bruksanvisning

SHOLKOFF BALLOON HYSTEROSALPINGOGRAPHY CATHETER

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or a properly licensed practitioner).

DEVICE DESCRIPTION

- Radiopaque polyvinylchloride catheter with silicone balloon
- Radiopaque TFE introducing sheath
- Syringe

INTENDED USE

Used for injection of contrast medium into the uterine cavity and fallopian tubes for evaluation of tubal patency. The introducing sheath facilitates placement of the catheter into the uterine cavity.

CONTRAINDICATIONS

None known

WARNINGS

Always inflate the balloon with a sterile liquid. Never inflate with air, carbon dioxide or any other gas.

PRECAUTIONS

- Do not overinflate the balloon. Using excessive pressure to inflate the balloon on this device can cause the balloon to rupture.
- Refer to the product label or the inflation check valve on the balloon device for appropriate balloon volume.
- The potential effects of phthalates on pregnant/nursing women or children have not been fully characterized and there may be concern for reproductive and developmental effects.

INSTRUCTIONS FOR USE

1. If desired, connect the balloon catheter to a disposable connecting tube.
2. Fill the assembled balloon catheter and connecting tube with contrast medium and remove any air bubbles.
3. Position a vaginal speculum within the vagina so the external cervical os can be visualized during catheter insertion.
4. Cleanse the cervix with an appropriate cleansing solution.
5. Place the distal portion of the catheter inside the introducing sheath.
6. Position the introducing sheath at the external cervical os.
7. Use the introducing sheath as a guide and, with a slight rotary motion, advance the catheter through the cervical canal and into the uterus.
8. Fill the syringe with sterile saline solution. Eliminate air from the syringe.
9. Inflate the balloon by instilling sterile saline solution and close the stopcock once desired volume is injected.
10. Pull back gently on the catheter shaft to seat the inflated balloon against the internal cervical os.
11. Inject sufficient contrast medium through the catheter into the uterus.
12. Perform fluoroscopic procedure.
13. Deflate the balloon.
14. Take a fluoroscopic final examination of the lower uterus.
15. Remove the catheter from the cervix when the examination is complete.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCE

S. Sholkoff: "Balloon Hysterosalpingography Catheter," *American Journal of Roentgenology*, 149 (1987), 995-996.

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© COPYRIGHT COOK 2012
COOK IRELAND LTD.