
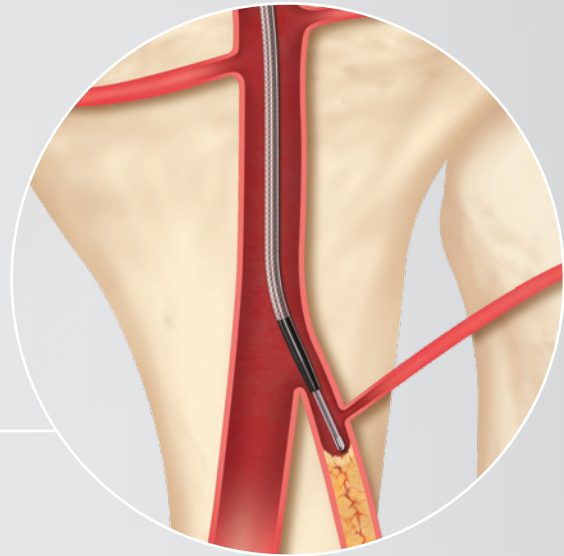


# Reach and cross lesions with the support catheter that is engineered for the periphery.

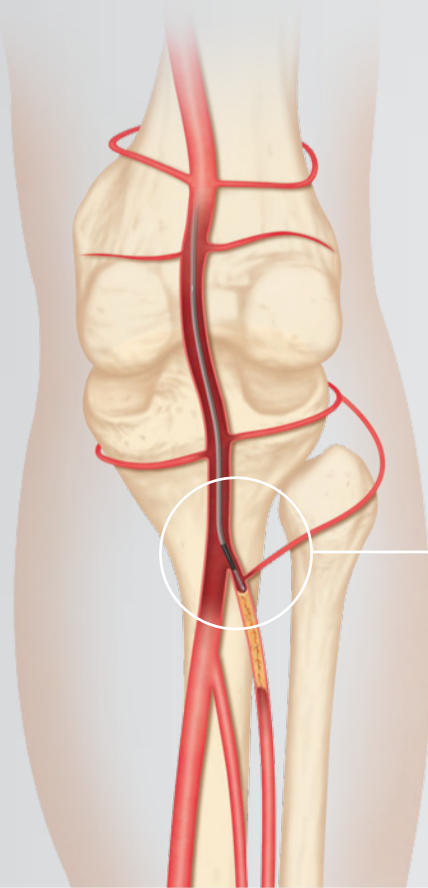

NOW  
AVAILABLE  
.014"  
Platform

**CXI<sup>®</sup>**  
SUPPORT CATHETER

## Exceptional Pushability

CXI's stainless steel braid provides a 1:1 torque response and exceptional pushability and kink-resistance. A hydrophilic coating on the distal 40 cm further enhances pushability and also increases trackability.

## Coaxial Capability

The 2.3 Fr and 2.6 Fr sizes can fit coaxially in the 4.0 Fr size to double the amount of support. The smaller sizes can also pass through the Micropuncture<sup>®</sup> Pedal Access Set, and can fit in any catheter that has an .035 inch lumen.

## Enhanced Visibility

Radiopaque bands at the tip and at 5, 10, and 15 cm enhance visualization and allow you to measure vessel segments. Exterior depth markers clearly show progression into the sheath.



## CXI SUPPORT CATHETER

CXI is now available in an .014 inch platform, pictured here, that features platinum-iridium marker bands for exceptional visibility.

Angled tips allow you to select branched vessels.


The seamless transition from the tip to the wire guide is designed to minimize vessel trauma.

Platinum-iridium marker bands are at the tip and at 5, 10, and 15 cm.

Exterior depth markers at 45, 55, 90, and 110 cm clearly show progression into the sheath.

A stainless steel braid provides 1:1 torque response and enhances pushability and kink-resistance.

The hydrophilic coating on the distal 40 cm increases pushability and trackability.

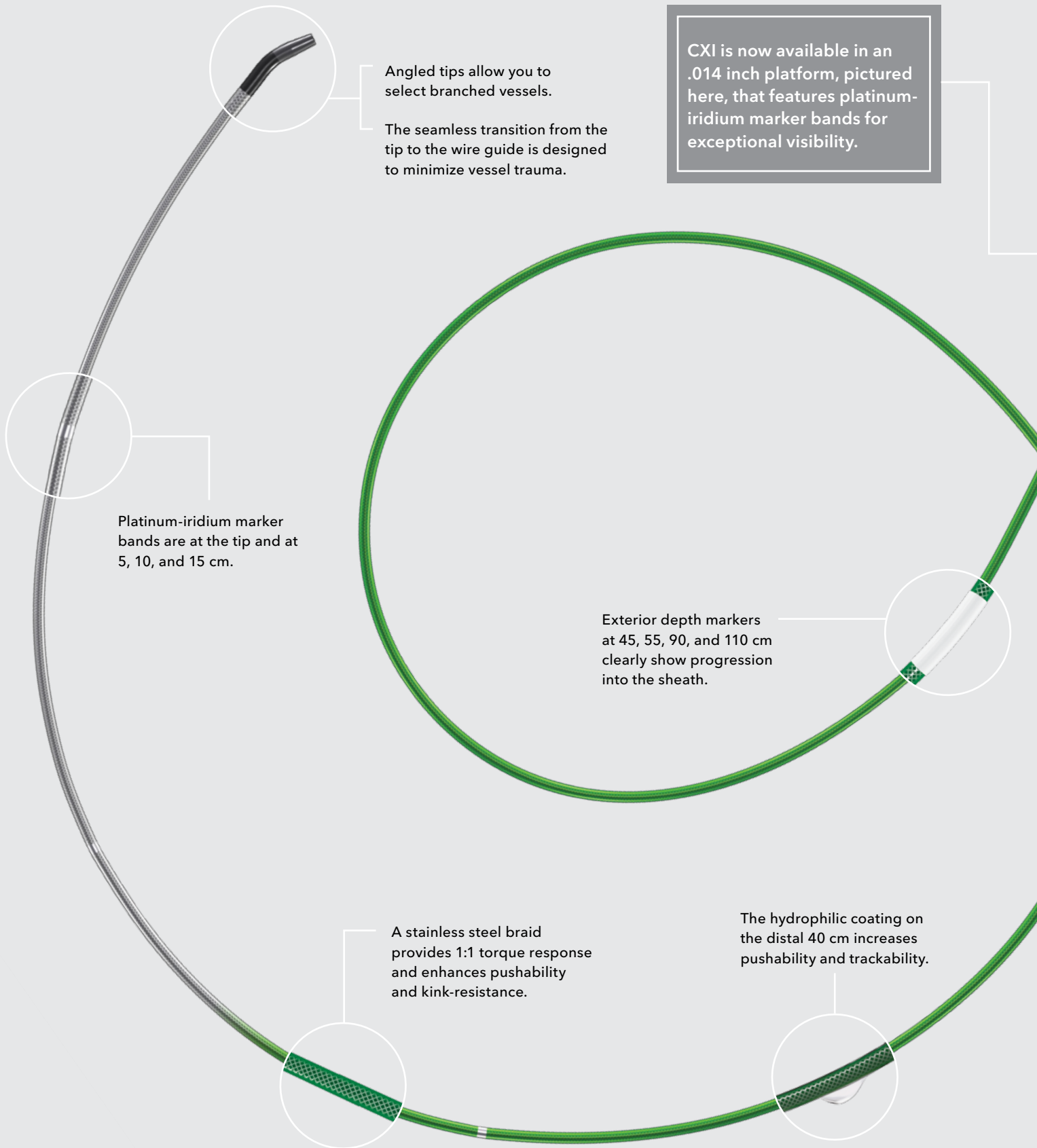


## CXI SUPPORT CATHETER


| Order Numbers                      | Reference Part Number   | Fr  | Accepts Wire Guide Diameter inch | Length cm | Tip Configuration |
|------------------------------------|-------------------------|-----|----------------------------------|-----------|-------------------|
| With platinum-iridium marker bands |                         |     |                                  |           |                   |
| G18369                             | CXI-2.3-14-65-0         | 2.3 | .014                             | 65        | Straight          |
| G18371                             | CXI-2.3-14-90-0         | 2.3 | .014                             | 90        | Straight          |
| G23160                             | CXI-2.3-14-135-0        | 2.3 | .014                             | 135       | Straight          |
| G18373                             | CXI-2.3-14-150-0        | 2.3 | .014                             | 150       | Straight          |
| G18370                             | CXI-2.3-14-65-ANG       | 2.3 | .014                             | 65        | ANG               |
| G18372                             | CXI-2.3-14-90-ANG       | 2.3 | .014                             | 90        | ANG               |
| G23162                             | CXI-2.3-14-135-ANG      | 2.3 | .014                             | 135       | ANG               |
| G18376                             | CXI-2.3-14-150-ANG      | 2.3 | .014                             | 150       | ANG               |
| G23858                             | CXI-2.3-14-65-ANG2      | 2.3 | .014                             | 65        | ANG2              |
| G23859                             | CXI-2.3-14-90-ANG2      | 2.3 | .014                             | 90        | ANG2              |
| G23860                             | CXI-2.3-14-135-ANG2     | 2.3 | .014                             | 135       | ANG2              |
| G23861                             | CXI-2.3-14-150-ANG2     | 2.3 | .014                             | 150       | ANG2              |
| With tungsten marker bands         |                         |     |                                  |           |                   |
| G18367                             | CXI-2.6-18-65-P-NS-0    | 2.6 | .018                             | 65        | straight          |
| G18368                             | CXI-2.6-18-65-P-NS-ANG  | 2.6 | .018                             | 65        | angled            |
| G50004                             | CXI-2.6-18-90-P-NS-0    | 2.6 | .018                             | 90        | straight          |
| G50002                             | CXI-2.6-18-90-P-NS-ANG  | 2.6 | .018                             | 90        | angled            |
| G50001                             | CXI-2.6-18-150-P-NS-0   | 2.6 | .018                             | 150       | straight          |
| G50003                             | CXI-2.6-18-150-P-NS-ANG | 2.6 | .018                             | 150       | angled            |
| G52546                             | CXI-4.0-35-90-P-NS-0    | 4.0 | .035                             | 90        | straight          |
| G52547                             | CXI-4.0-35-90-P-NS-DAV  | 4.0 | .035                             | 90        | angled            |
| G52542                             | CXI-4.0-35-135-P-NS-0   | 4.0 | .035                             | 135       | straight          |
| G52543                             | CXI-4.0-35-135-P-NS-DAV | 4.0 | .035                             | 135       | angled            |
| G52544                             | CXI-4.0-35-150-P-NS-0   | 4.0 | .035                             | 150       | straight          |
| G52545                             | CXI-4.0-35-150-P-NS-DAV | 4.0 | .035                             | 150       | angled            |

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

### Tip Options

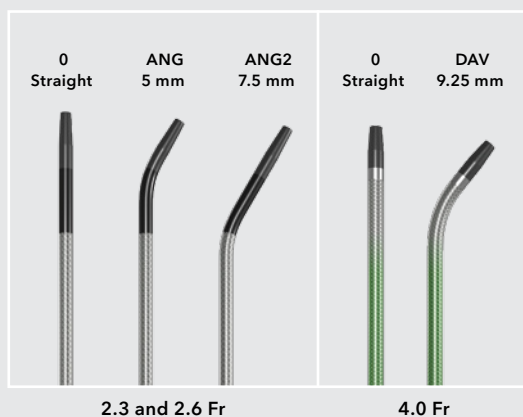


### Coaxial Capability

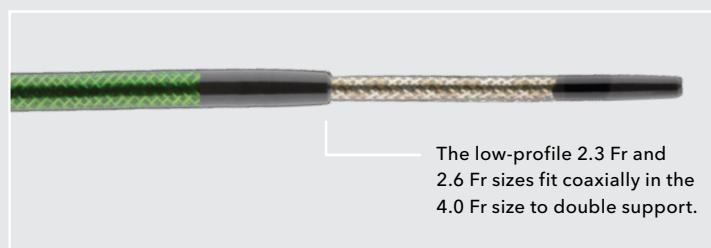

## Compatible Products




The four tip-load options of the **Approach® CTO Microwire Guide** can help you cross both challenging lesions and chronic total occlusions in the legs.

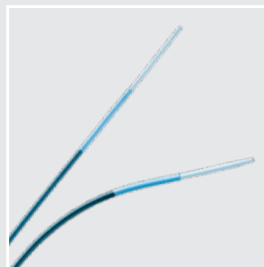

The **Flexor® Ansel Guiding Sheath with High-Flex Dilator** features our most advanced technologies, which enhance pushability and trackability, and is available in 4-8, 10, and 12 Fr.



The **Micropuncture® Pedal Access Set** can facilitate a retrograde tibiopedal approach to give you another chance at limb salvage.\*

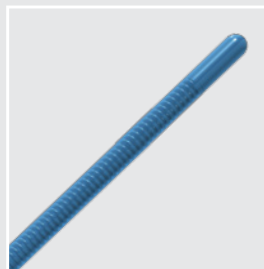

The hydrophilic tip and supportive shaft of the **Approach Hydro ST Microwire Guide** can help you target anatomy that is tortuous or below the knee.

\* Montero-Baker M, Schmidt A, Bräunlich S, et al. Retrograde approach for complex popliteal and tibioperoneal occlusions. *J Endovasc Ther.* 2008;15(5):594-604.

### Customer Service

**EMEA:** EDI - [www.cookmedical.com/edi.do](http://www.cookmedical.com/edi.do)  
**Distributors:** +353 61239240, [ssc.distributors@cookmedical.com](mailto:ssc.distributors@cookmedical.com)  
**Austria:** +43 179567121, [oe.orders@cookmedical.com](mailto:oe.orders@cookmedical.com)  
**Belgium:** +32 27001633, [be.orders@cookmedical.com](mailto:be.orders@cookmedical.com)  
**Denmark:** +45 38487607, [da.orders@cookmedical.com](mailto:da.orders@cookmedical.com)  
**Finland:** +358 972519996, [fi.orders@cookmedical.com](mailto:fi.orders@cookmedical.com)  
**France:** +33 171230269, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
**Germany:** +49 6950072804, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
**Hungary:** +36 17779199, [hu.orders@cookmedical.com](mailto:hu.orders@cookmedical.com)  
**Ireland:** +353 61239252, [ie.orders@cookmedical.com](mailto:ie.orders@cookmedical.com)  
**Italy:** +39 0269682853, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
**Netherlands:** +31 202013367, [nl.orders@cookmedical.com](mailto:nl.orders@cookmedical.com)  
**Norway:** +47 23162968, [no.orders@cookmedical.com](mailto:no.orders@cookmedical.com)  
**Spain:** +34 912702691, [es.orders@cookmedical.com](mailto:es.orders@cookmedical.com)  
**Sweden:** +46 858769468, [se.orders@cookmedical.com](mailto:se.orders@cookmedical.com)  
**Switzerland - French:** +41 448009609, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
**Switzerland - Italian:** +41 448009609, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
**Switzerland - German:** +41 448009609, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
**United Kingdom:** +44 2073654183, [uk.orders@cookmedical.com](mailto:uk.orders@cookmedical.com)

[www.cookmedical.com](http://www.cookmedical.com)

**Americas:** EDI - [www.cookmedical.com/edi.do](http://www.cookmedical.com/edi.do)  
**Phone:** +1 812.339.2235, 800.457.4500, Fax: 800.554.8335  
**E-mail:** [orders@cookmedical.com](mailto:orders@cookmedical.com)

**Australia:**  
**Phone:** +61 738411188, 1800777222, Fax: +61 738411288, 1800077283  
**E-mail:** [cau.custserv@cookmedical.com](mailto:cau.custserv@cookmedical.com)

