
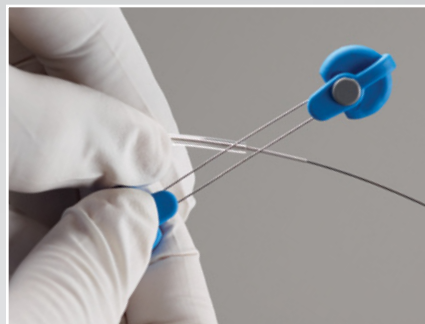


# Maximize the rail effect with proximal lead control.

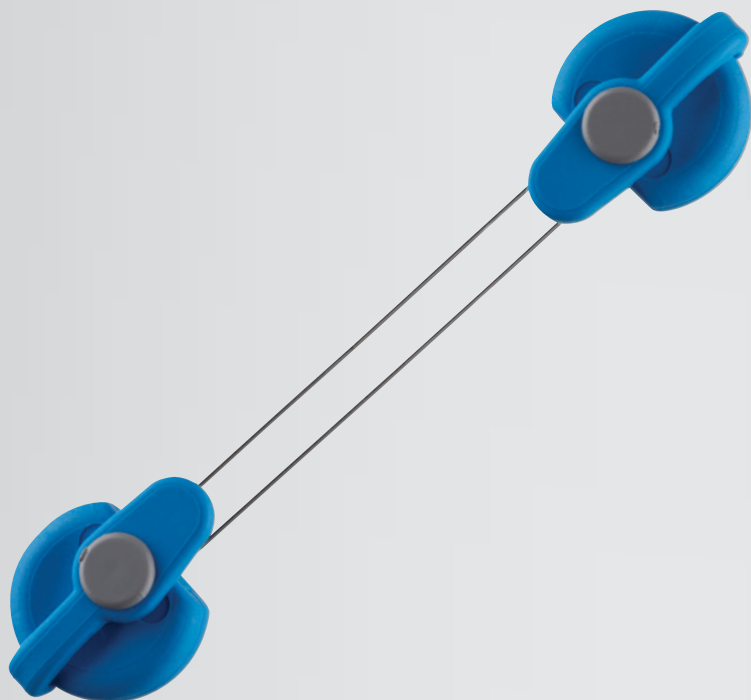


One-Tie™  
COMPRESSION COIL

The One-Tie™ Compression Coil is designed to make Lead Extraction™ procedures more manageable by securely binding together the Cook Medical locking stylet<sup>1</sup> and the proximal components of the targeted cardiac lead. The result is total lead control that can be achieved only by actively engaging a lead both proximally and distally. Total lead control is essential in order to create a consistent, transvenous guiding rail for extraction sheaths.

<sup>1</sup>One-Tie should be used only with the Liberator® Beacon® Tip Locking Stylet.



[www.cookmedical.com](http://www.cookmedical.com)

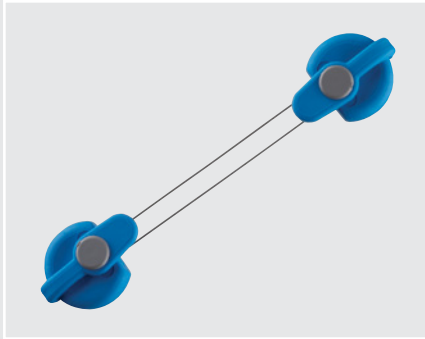

## One-Tie™

COMPRESSION COIL

When using a One-Tie Compression Coil, you will find that a troublesome and ineffective suture tie is no longer required. The simple-to-apply One-Tie can be used on nearly all of the cardiac, defibrillator, and pacemaker leads you may need to extract. The One-Tie Compression Coil will help make your extraction procedures less complex.

Order Number	Reference Part Number	Quantity
G31929	LR-OTE-N	5

One-Tie should be used only with the Liberator Beacon Tip Locking Stylet.  
Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

### Customer Service

**EMEA:** EDI - [www.cookmedical.com/edi.do](http://www.cookmedical.com/edi.do)  
 Distributors: +353 61239240, [ssc.distributors@cookmedical.com](mailto:ssc.distributors@cookmedical.com)  
 Austria: +43 179567121, [oe.orders@cookmedical.com](mailto:oe.orders@cookmedical.com)  
 Belgium: +32 27001633, [be.orders@cookmedical.com](mailto:be.orders@cookmedical.com)  
 Denmark: +45 38487607, [da.orders@cookmedical.com](mailto:da.orders@cookmedical.com)  
 Finland: +358 972519996, [fi.orders@cookmedical.com](mailto:fi.orders@cookmedical.com)  
 France: +33 171230269, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
 Germany: +49 6950072804, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
 Hungary: +36 17779199, [hu.orders@cookmedical.com](mailto:hu.orders@cookmedical.com)  
 Ireland: +353 61239252, [ie.orders@cookmedical.com](mailto:ie.orders@cookmedical.com)  
 Italy: +39 0269682853, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
 Netherlands: +31 202013367, [nl.orders@cookmedical.com](mailto:nl.orders@cookmedical.com)  
 Norway: +47 23162968, [no.orders@cookmedical.com](mailto:no.orders@cookmedical.com)  
 Spain: +34 912702691, [es.orders@cookmedical.com](mailto:es.orders@cookmedical.com)  
 Sweden: +46 858769468, [se.orders@cookmedical.com](mailto:se.orders@cookmedical.com)  
 Switzerland - French: +41 448009609, [fr.orders@cookmedical.com](mailto:fr.orders@cookmedical.com)  
 Switzerland - Italian: +41 448009609, [it.orders@cookmedical.com](mailto:it.orders@cookmedical.com)  
 Switzerland - German: +41 448009609, [de.orders@cookmedical.com](mailto:de.orders@cookmedical.com)  
 United Kingdom: +44 2073654183, [uk.orders@cookmedical.com](mailto:uk.orders@cookmedical.com)

**Americas:** EDI - [www.cookmedical.com/edi.do](http://www.cookmedical.com/edi.do)  
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335  
 E-mail: [orders@cookmedical.com](mailto:orders@cookmedical.com)

**Australia:**  
 Phone: +61 738411188, 1800777222, Fax: +61 738411288, 1800077283  
 E-mail: [cau.custserv@cookmedical.com](mailto:cau.custserv@cookmedical.com)



[www.cookmedical.com](http://www.cookmedical.com)