

# Products for Dialysis




**COOK<sup>®</sup>**

# Contents

<b>Subclavian Double Lumen Hemodialysis Sets and Trays</b> .....	<b>2</b>
<b>Femoral Hemodialysis Set</b> .....	<b>3</b>
<b>Uldall Double Lumen Hemodialysis Catheter Trays</b> .....	<b>4-5</b>
<b>Pediatric Acute Peritoneal Dialysis Set</b> .....	<b>6</b>
<b>Acute Peritoneal Dialysis Sets and Trays</b> .....	<b>7</b>
<b>Tenckhoff Acute Peritoneal Dialysis Catheters and Sets - One Cuff</b> .....	<b>8</b>
<b>Spiral Acute Peritoneal Dialysis Catheters and Set - One Cuff - Preperitoneal Cuff Position</b> .....	<b>9</b>
<b>Tenckhoff Chronic Peritoneal Dialysis Catheters and Sets - Two Cuffs</b> .....	<b>10</b>
<b>Spiral Chronic Peritoneal Dialysis Catheters and Set - Two Cuffs</b> .....	<b>11</b>
<b>Stiffening Stylets for Acute and Chronic Peritoneal Dialysis Catheters</b> .....	<b>12</b>
<b>References</b> .....	<b>13</b>
<b>Acknowledgement</b> .....	<b>13</b>
<b>Videotapes</b> .....	<b>13</b>


## SUBCLAVIAN DOUBLE LUMEN HEMODIALYSIS SETS AND TRAYS

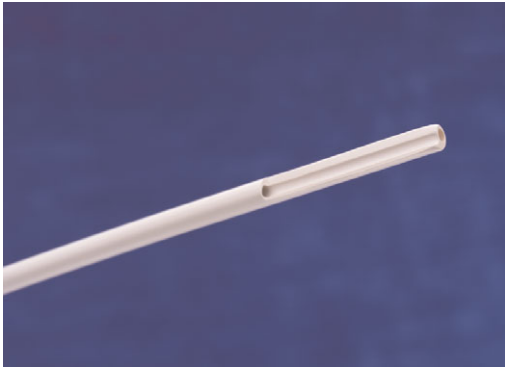

Used for blood withdrawal and infusion.

- The double lumen construction allows blood withdrawal through the proximal port and infusion through the distal port.
- The Peel-Away® sheath allows the catheter to be introduced percutaneously.
- The swivel fitting allows the catheter tip to be rotated within the vessel without removing the tie-down sutures.

Supplied sterile in peel-open packages. Intended for one-time use.

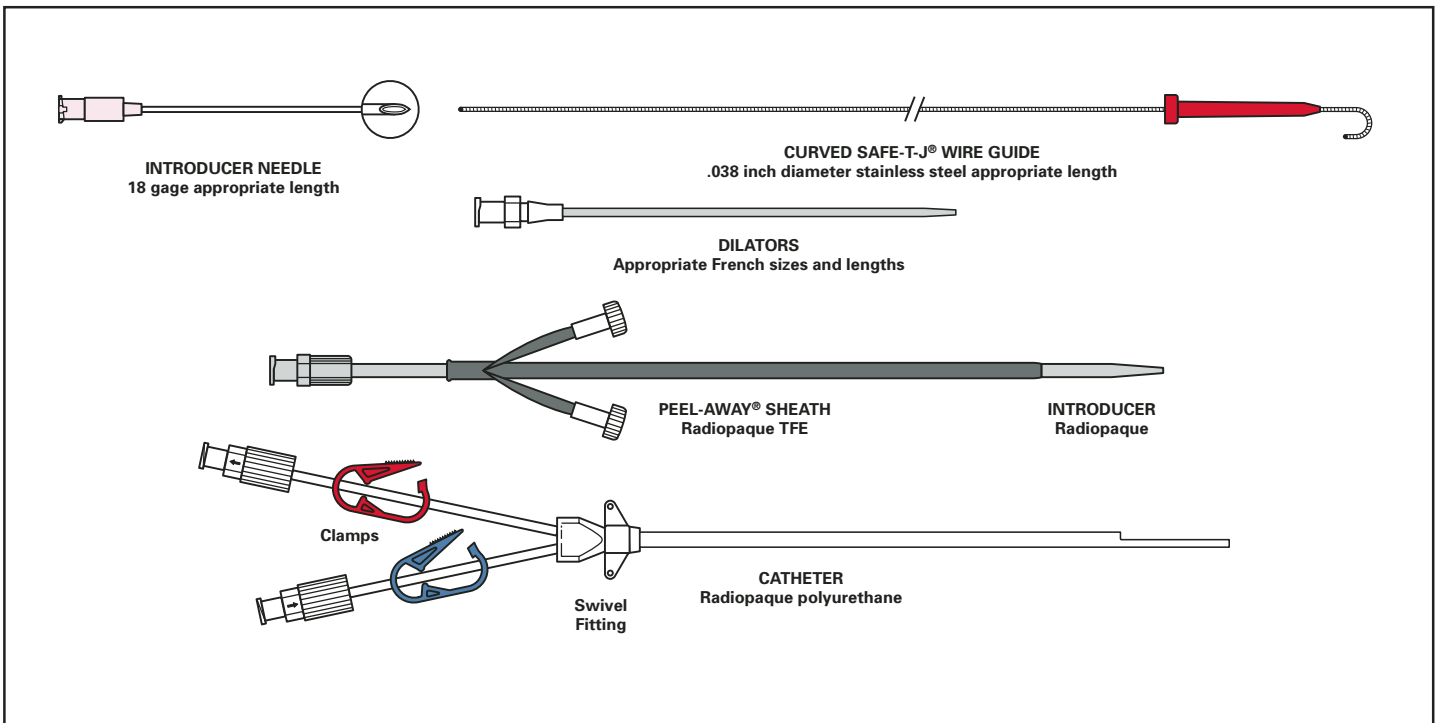


ORDER NUMBER	French Size	CATHETER		SHEATH		INTRODUCER		Quick Reorder Number <sup>1</sup>		
		Arterial (Red) Withdrawal Lumen Diameter	Arterial (Red) Lumen Volume	Venous (Blue) Infusion Lumen Diameter	Venous (Blue) Lumen Volume	Length	Length		French Size	Length
<b>SETS</b>										
C-DDS-1001U-10	10.0	.048 inch	.3 cc	.048 inch	.4 cc	10 cm	9 cm	11.0	13 cm	155756
C-DDS-1201U-15	12.0	.066 inch	.6 cc	.066 inch	.7 cc	15 cm	15.5 cm	12.5	20 cm	155757
C-DDS-1201U-20	12.0	.066 inch	.8 cc	.066 inch	.9 cc	20 cm	15.5 cm	12.5	20 cm	155760
<b>TRAYS<sup>2</sup></b>										
C-DDSY-1001U-10	10.0	.048 inch	.3 cc	.048 inch	.4 cc	10 cm	9 cm	11.0	13 cm	155958
C-DDSY-1201U-15	12.0	.066 inch	.6 cc	.066 inch	.7 cc	15 cm	15.5 cm	12.5	20 cm	155758
C-DDSY-1201U-20	12.0	.066 inch	.8 cc	.066 inch	.9 cc	20 cm	15.5 cm	12.5	20 cm	155761

U.S. Patent Numbers RE 31,855, 4,581,025; Canadian Patented 1987; Australian Patent Number 572,904; European Patent Number 0,162,982


<sup>1</sup>The Quick Reorder Number is for U.S.A. domestic use only.

<sup>2</sup>All trays consist of items shown above and the following items: 25 gage needle, 22 gage needle, lidocaine, povidone-iodine ointment, povidone-iodine swabsticks, fenestrated drape, gauze sponges, disposable syringes, disposable scalpel, suture with needle, needle holder, CSR wrap and prep tray.

## FEMORAL HEMODIALYSIS SET

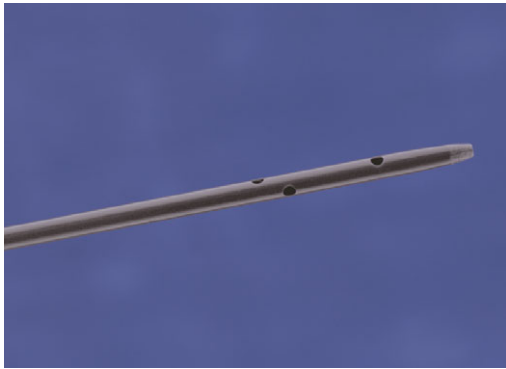


Used for blood withdrawal and infusion. Supplied sterile in peel-open packages. Intended for one-time use.

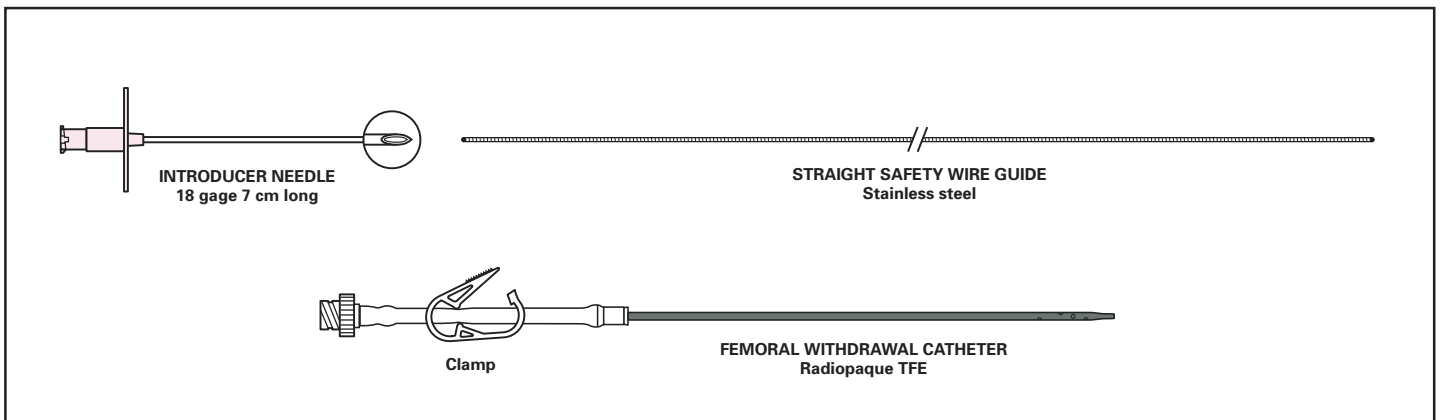


ORDER NUMBER	CATHETER			WIRE GUIDE		Remarks	Quick Reorder Number <sup>1</sup>
	French Size	Length	Side-ports	Diameter	Length		
<b>C-HDS-700T</b>	7.0	13 cm	5	.038 inch	50 cm	Femoral withdrawal catheter	<b>074751</b>

<sup>1</sup>The Quick Reorder Number is for U.S.A. domestic use only.

### ULDALL DOUBLE LUMEN HEMODIALYSIS CATHETER TRAYS

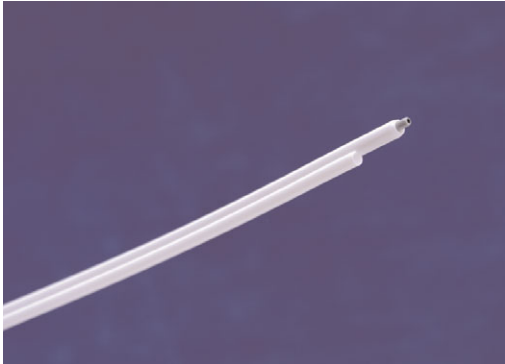

Used for vascular access. Supplied sterile in peel-open packages. Intended for one-time use.

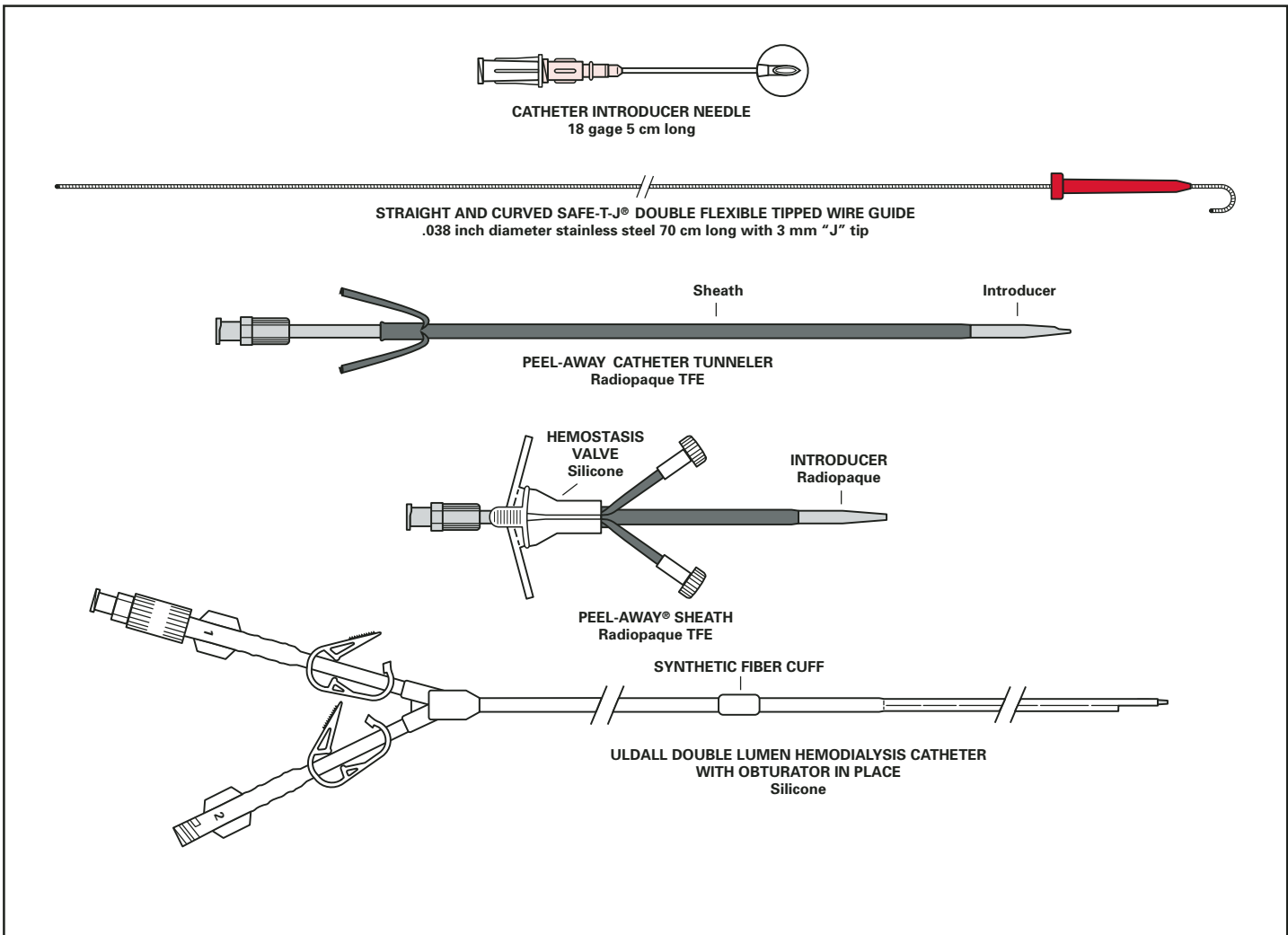


## ULDALL DOUBLE LUMEN HEMODIALYSIS CATHETER TRAYS

ORDER NUMBER	CATHETER				Quick Reorder Number <sup>1</sup>
	French Size	#1 Lumen Diameter	#2 Lumen Diameter	Length	
<b>TRAYS</b>					
<b>Pediatric</b>					
<b>C-DDSCY-1100-18-5.0-ULDALL</b>	11.0	.051 inch	.051 inch	18 cm	<b>256481</b>
<b>Adult</b>					
<b>C-DDSCY-1600-26-5.0-ULDALL-CW</b>	16.0	.080 inch	.080 inch	26 cm	<b>234457</b>


Trays also include: **11 French** - 25 gage needle, 22 gage needle, needle holder, lidocaine, povidone-iodine swabsticks, fenestrated drape, gauze sponges, disposable syringes, disposable scalpel, suture with needle, CSR wrap and prep tray; **16 French** - 25 gage needle, 19 gage introducer needle, needle holder, lidocaine, povidone-iodine swabsticks, fenestrated drape, gauze sponges, disposable syringes, disposable scalpel, suture with needle, CSR wrap and prep tray.

<sup>1</sup>The Quick Reorder Number is for U.S.A. domestic use only.

### PEDIATRIC ACUTE PERITONEAL DIALYSIS SET

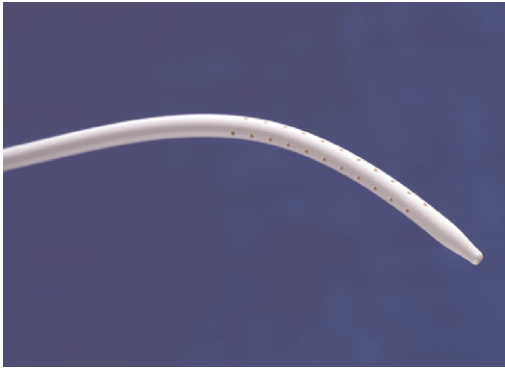


Used to administer dialysis solution into the peritoneal cavity. Supplied sterile in peel-open packages. Intended for one-time use.

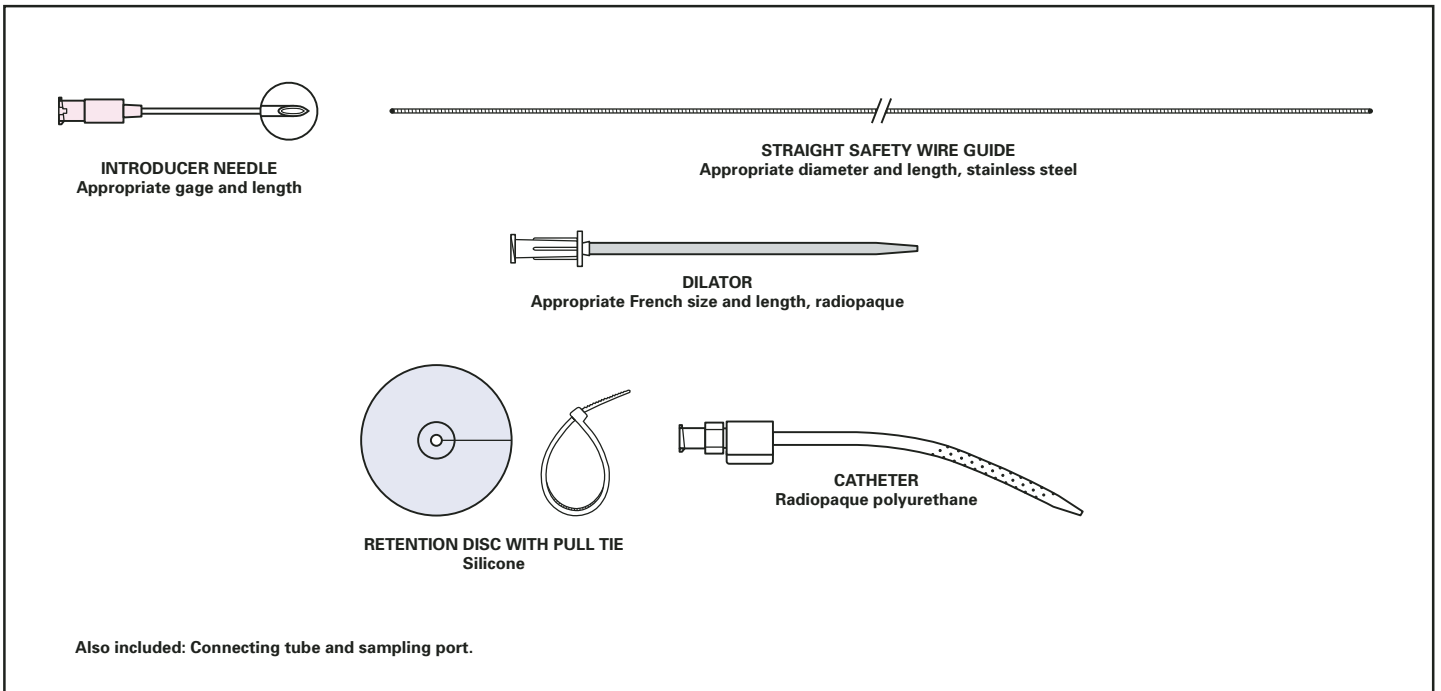


ORDER NUMBER	CATHETER			WIRE GUIDE		NEEDLE	CONNECTING TUBE	Quick Reorder Number <sup>1</sup>
	French Size	Length	Side-Ports	Diameter	Length	Gage	Length	
<b>SET</b> <b>C-PDS-851U-PT</b>	8.5	8 cm	44	.035 inch	30 cm	18T	30 cm	<b>176980</b>

<sup>1</sup>The Quick Reorder Number is for U.S.A. domestic use only.

## ACUTE PERITONEAL DIALYSIS SETS AND TRAYS

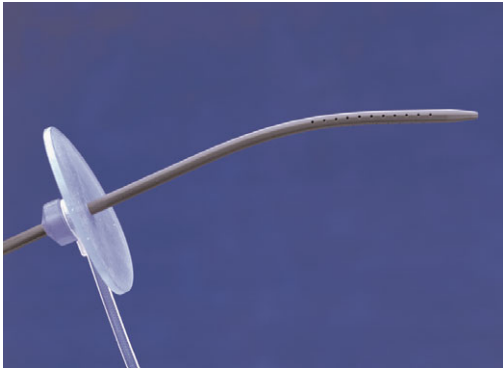

Used to administer dialysis solution into the peritoneal cavity. Supplied sterile in peel-open packages. Intended for one-time use.

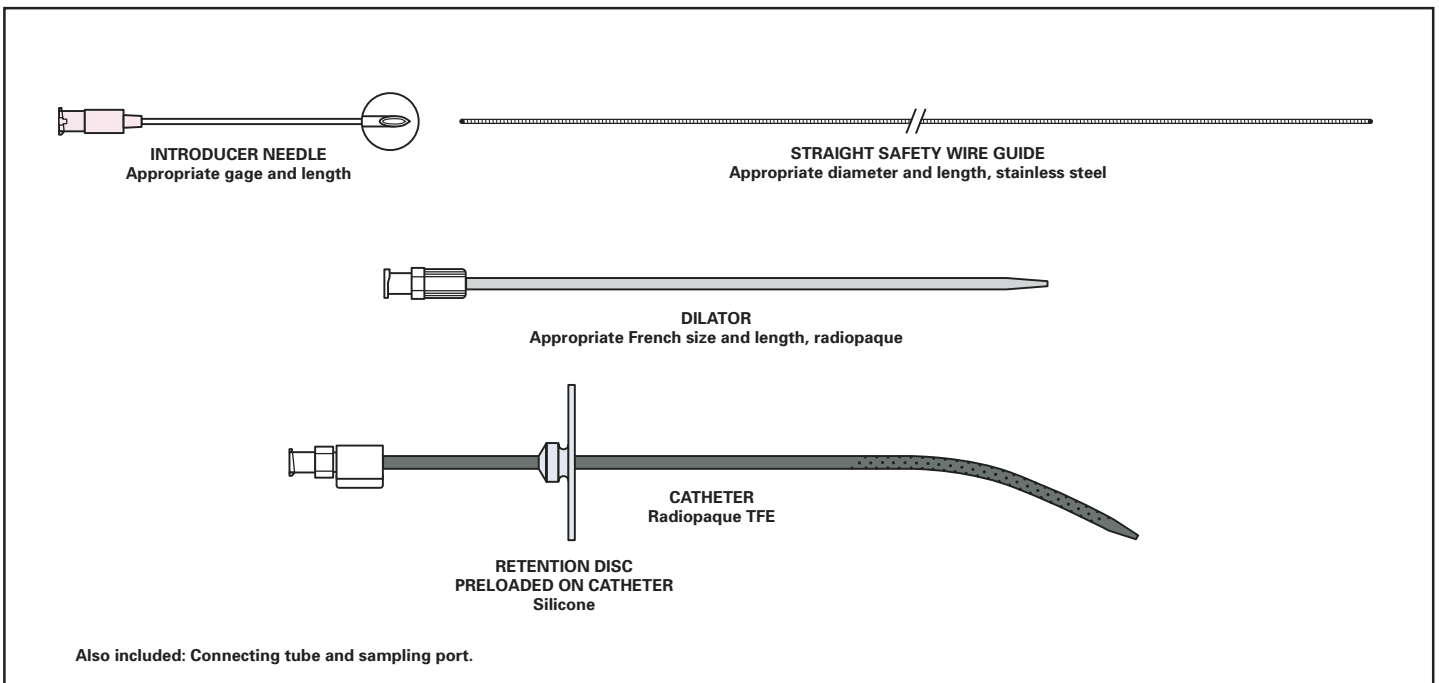


ORDER NUMBER	CATHETER			WIRE GUIDE		NEEDLE	CONNECTING TUBE	Quick Reorder Number <sup>1</sup>
	French Size	Length	Side-Ports	Diameter	Length	Gage	Length	
<b>PEDIATRIC Set</b>								
<b>C-PDS-901T-15</b>	9.0	15 cm	44	.038 inch	40 cm	18T	30 cm	<b>155767</b>
<b>Tray<sup>2</sup></b>								
<b>C-PDSY-901T-15</b>	9.0	15 cm	44	.038 inch	40 cm	18T	30 cm	<b>155769</b>
<b>ADULT Set</b>								
<b>C-PDS-1101T-20</b>	11.0	20 cm	90	.052 inch	60 cm	15T	30 cm	<b>155765</b>
<b>Tray<sup>2</sup></b>								
<b>C-PDSY-1101T-20</b>	11.0	20 cm	90	.052 inch	60 cm	15T	30 cm	<b>155766</b>

<sup>1</sup>The Quick Reorder Number is for U.S.A. domestic use only

<sup>2</sup>All trays consist of items shown above and the following items: 25 gage needle, 22 gage needle, needle holder, lidocaine, povidone-iodine swabsticks, povidone-iodine ointment, fenestrated drape, gauze sponges, disposable syringes, disposable scalpel, suture with needle, CSR wrap and prep tray.




## TENCKHOFF ACUTE PERITONEAL DIALYSIS CATHETERS AND SETS ONE CUFF

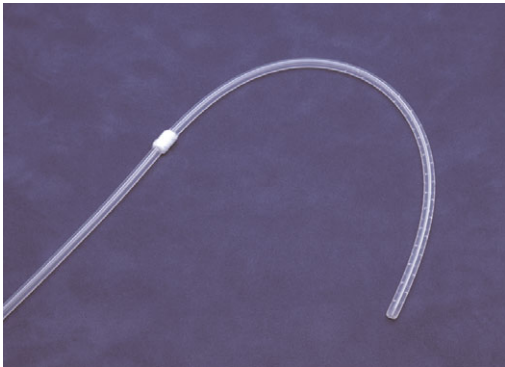

Used for acute access to the peritoneal cavity.


- Peel-Away® sheath allows the catheter to be introduced percutaneously.
  - Synthetic fiber cuff is affixed to catheter to allow tissue ingrowth.
  - Sideports are spiraled.
- Supplied sterile in peel-open packages. Intended for one-time use.

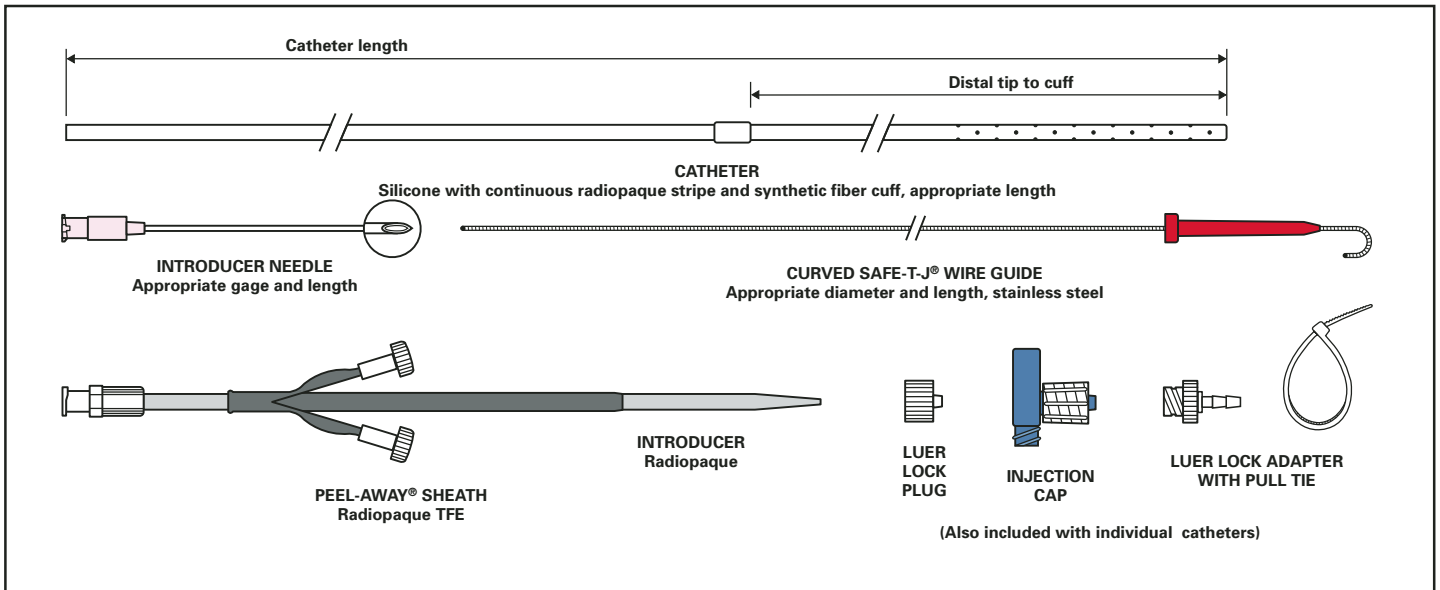

ORDER NUMBER	French Size	Length	CATHETER		Distance From Distal Tip to Cuff	SHEATH		INTRUDCER		Quick Reorder Number <sup>1</sup>
			Side-ports	Length		Length	French Size	Length		
<b>CATHETERS</b>										
<b>Subcutaneous Cuff Position</b>										
C-TAP-9.5-37-20-O-16.5	9.5	37 cm	20		16.5 cm					221489
C-TAP-15-31-12-O-10.5	15.0	31 cm	12		10.5 cm					221466
C-TAP-15-37-20-O-16.5	15.0	37 cm	20		16.5 cm					221468
C-TAP-15-42-28-O-21	15.0	42 cm	28		21 cm					221446
<b>SETS</b>										
<b>Subcutaneous Cuff Position</b>										
C-TAPS-9.5-31-12-O-10.5	9.5	31 cm	12		10.5 cm	10 cm	11.0	17 cm		223253
C-TAPS-9.5-37-20-O-16.5	9.5	37 cm	20		16.5 cm	10 cm	11.0	17 cm		221531
C-TAPS-15-31-12-O-10.5	15.0	31 cm	12		10.5 cm	10 cm	16.0	17 cm		221542
C-TAPS-15-37-20-O-16.5	15.0	37 cm	20		16.5 cm	10 cm	16.0	17 cm		221500
C-TAPS-15-42-28-O-21	15.0	42 cm	28		21 cm	15.5 cm	16.0	20 cm		221514
C-TAPS-15-46-60-O-25	15.0	46 cm	60		25 cm	15.5 cm	16.0	20 cm		221594

<sup>1</sup>The Quick Reorder Number is for U.S.A. domestic use only.


## SPIRAL ACUTE PERITONEAL DIALYSIS CATHETERS AND SET ONE CUFF - PREPERITONEAL CUFF POSITION

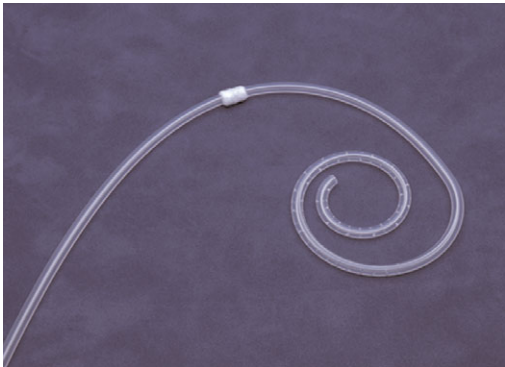

Used for acute access to the peritoneal cavity.

- Peel-Away® sheath allows the catheter to be introduced percutaneously.
- Synthetic fiber cuff is affixed to catheter to allow tissue ingrowth.
- Sideports are spiraled.


Supplied sterile in peel-open packages. Intended for one-time use.

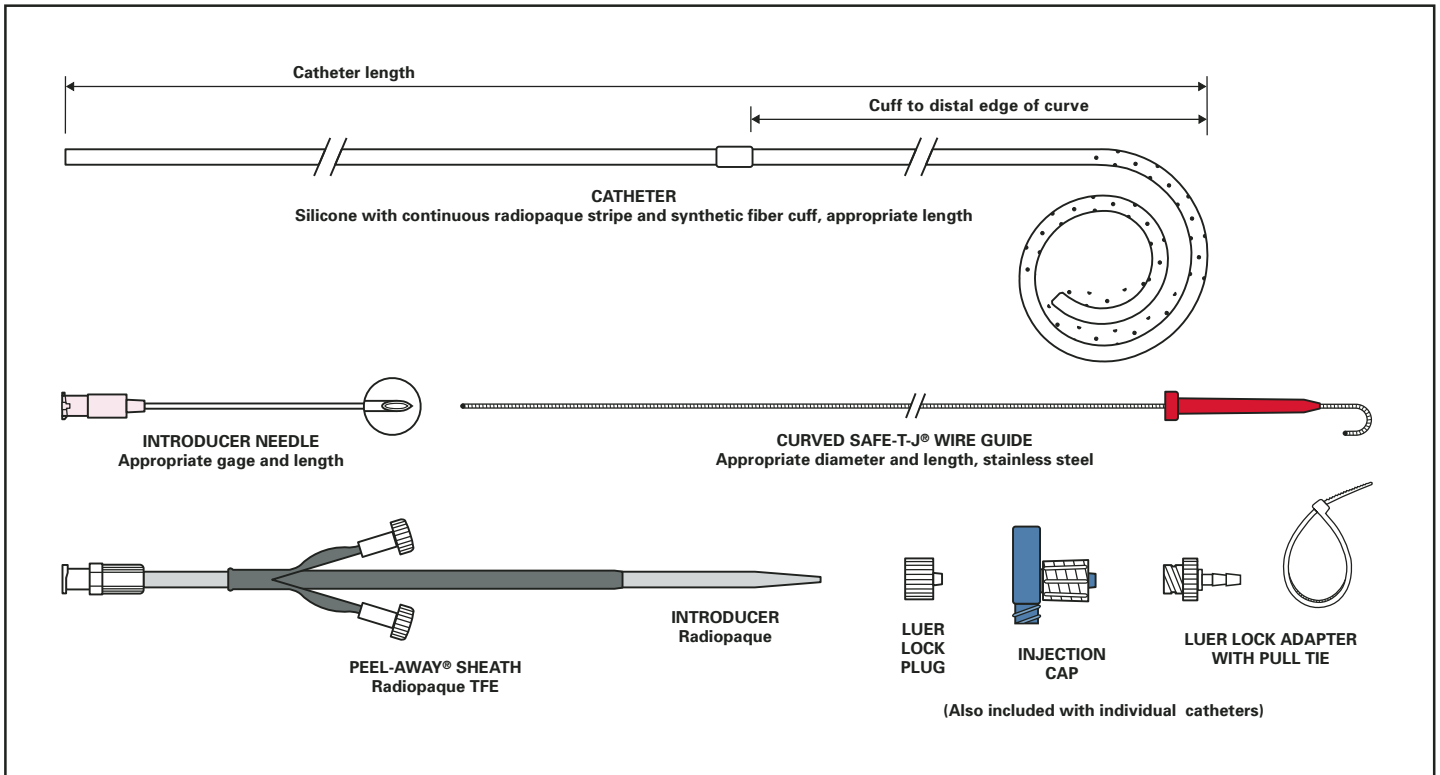

ORDER NUMBER	CATHETER				SHEATH		INTRODUCER		Quick Reorder Number <sup>1</sup>
	French Size	Length	Side-ports	Cuff to Distal Edge of Curve	Length	French Size	Length		
<b>CATHETERS</b>									
C-TAP-15-39-78-SPRL-14	15.0	39 cm	78	14 cm					225848
C-TAP-15-35-78-SPRL-10	15.0	35 cm	78	10 cm					225843
<b>SET</b>									
C-TAPS-15-39-78-SPRL-14	15.0	39 cm	78	14 cm	10 cm	16.0	17 cm		225666

<sup>1</sup>The Quick Reorder Number is for U.S.A. domestic use only.


## TENCKHOFF CHRONIC PERITONEAL DIALYSIS CATHETERS AND SETS TWO CUFFS

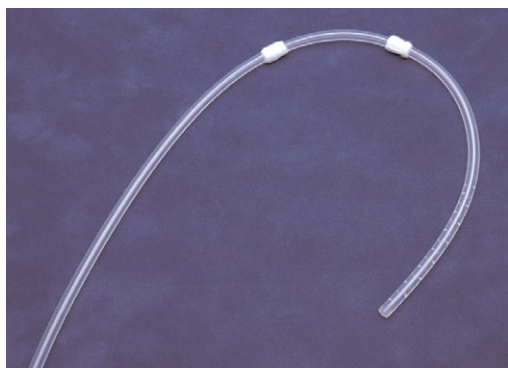

Used for chronic access to the peritoneal cavity.

- Peel-Away® sheath allows the catheter to be introduced percutaneously.
- Synthetic fiber cuff is affixed to catheter to allow tissue ingrowth.
- Sideports are spiraled.


Supplied sterile in peel-open packages. Intended for one-time use.

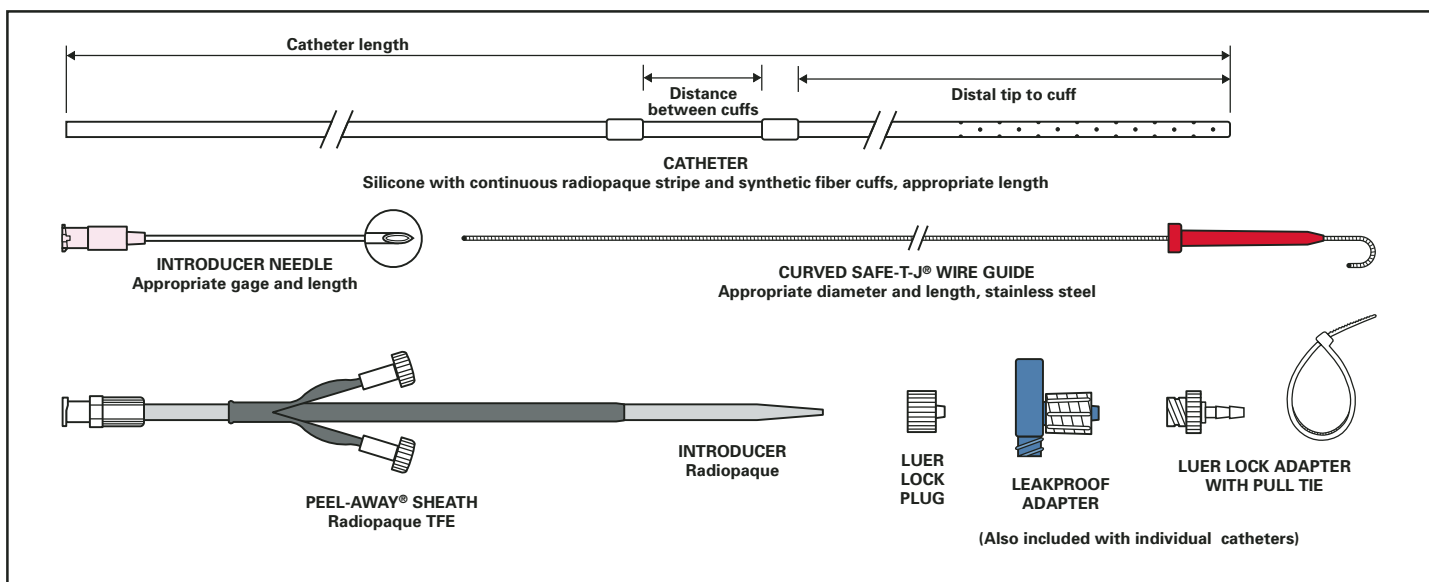

ORDER NUMBER	CATHETER					SHEATH	INTRODUCER		Quick Reorder Number <sup>1</sup>
	French Size	Length	Side-ports	Distance From Distal Tip to Cuff	Distance Between Cuffs	Length	French Size	Length	
<b>CATHETERS</b>									
C-TCP-9.5-37-20-O-11-5	9.5	37 cm	20	11 cm	5 cm				221456
C-TCP-15-42-28-O-15-5	15.0	42 cm	28	15 cm	5 cm				221539
C-TCP-15-46-60-O-19-5	15.0	46 cm	60	19 cm	5 cm				223380
<b>SETS</b>									
C-TCPS-9.5-37-20-O-11-5	9.5	37 cm	20	11 cm	5 cm	10 cm	11.0	17 cm	221470
C-TCPS-15-31-12-O-7-3	15.0	31 cm	12	7 cm	3 cm	10 cm	16.0	17 cm	221568
C-TCPS-15-42-28-O-15-5	15.0	42 cm	28	15 cm	5 cm	15.5 cm	16.0	20 cm	221496

<sup>1</sup>The Quick Reorder Number is for U.S.A. domestic use only.


## SPIRAL CHRONIC PERITONEAL DIALYSIS CATHETERS AND SET TWO CUFFS

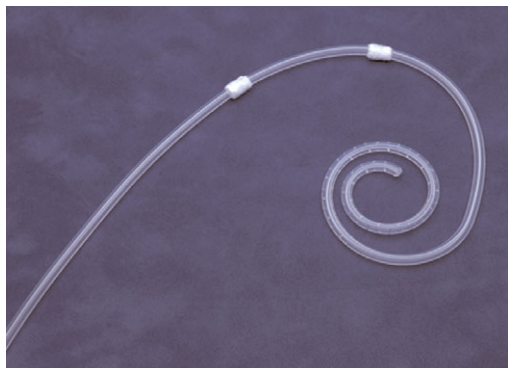

Used for chronic access to the peritoneal cavity.

- Peel-Away® sheath allows the catheter to be introduced percutaneously.
- Synthetic fiber cuff is affixed to catheter to allow tissue ingrowth.
- Sideports are spiraled.


Supplied sterile in peel-open packages. Intended for one-time use.

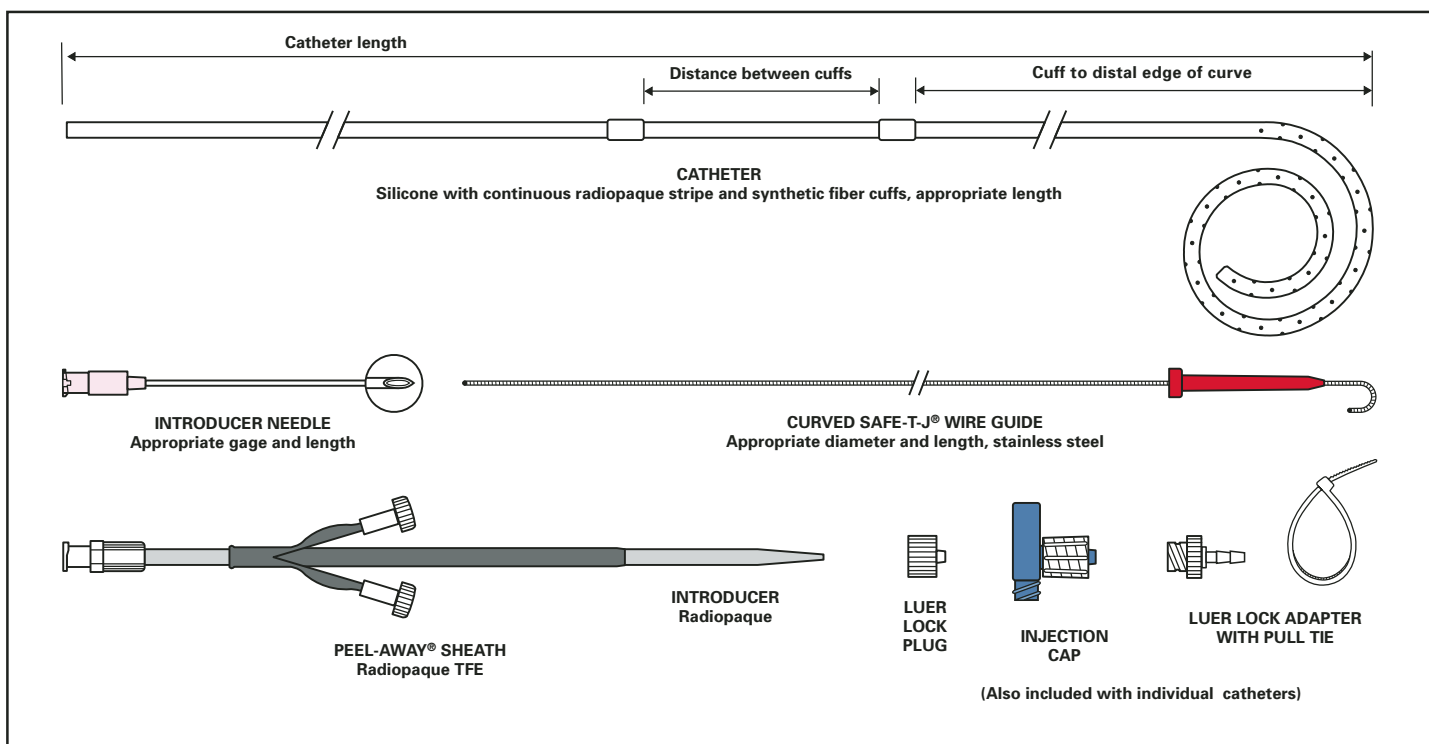


ORDER NUMBER	CATHETER				Distance Between Cuffs	SHEATH		INTRODUCER		Quick Reorder Number <sup>1</sup>
	French Size	Length	Side-ports	Length		French Size	Length			
<b>CATHETERS</b>										
C-TCP-15-41-78-SPRL-14-6	15.0	41 cm	78	6 cm						225917
C-TCP-15-35-78-SPRL-10-4	15.0	35 cm	78	4 cm						225909
<b>SET</b>										
C-TCPS-15-41-78-SPRL-14-6	15.0	41 cm	78	6 cm	15.5 cm	16.0	20 cm			225899

<sup>1</sup>The Quick Reorder Number is for U.S.A. domestic use only.

## STIFFENING STYLETS FOR ACUTE AND CHRONIC PERITONEAL DIALYSIS CATHETERS

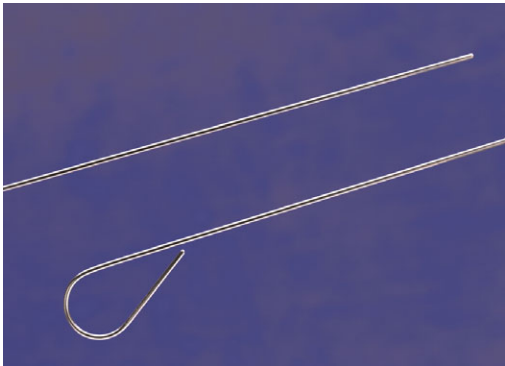

Used to facilitate introduction of acute and chronic peritoneal dialysis catheters. Supplied sterile in peel-open packages.

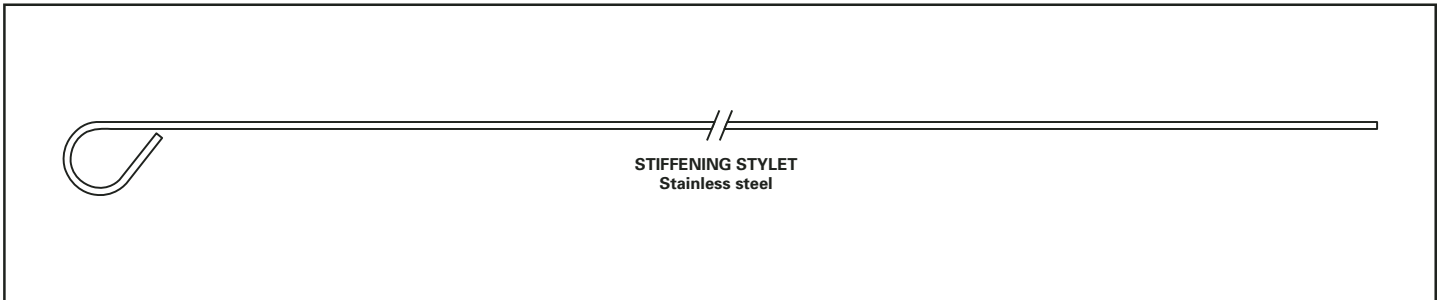

ORDER NUMBER	Diameter	Length	Remarks	Quick Reorder Number <sup>1</sup>
C-TCS-52-37	.052 inch	37 cm	Used with 9.5 French Tenckhoff catheter	136980
C-TCS-90-62	.090 inch	62 cm	Used with 15.0 French Tenckhoff catheter	093363

<sup>1</sup>The Quick Reorder Number is for U.S.A. domestic use only.

## REFERENCES

B. D. Nidus, A. Neusy: "Chronic Hemodialysis by Repeated Femoral Vein Cannulation," *Nephron*, 29 (1981), 195-197.

T. Nakanishi, M. Yanase, M. Fujii, Y. Tanaka, Y. Orita, H. Abe: "New Acute Peritoneal Dialysis Technique: Wire-Guide Insertion and Long-Term Indwelling of Peritoneal Catheter," *Nephron*, 37 (1984), 128-132.

S. Perras, A. R. Zappacosta, M. Mattern: "Comparison of Two Techniques for Percutaneous Peritoneal Dialysis Catheter Placement," *American Nephrology Nurses Association*, 12 (1985), 307-310.

J. L. Murphy, V. M. Reznik, S. Mendoza, B. Peterson, A. Rodarte, W. R. Griswold: "Use of a Guide-Wire Inserted Catheter for Acute Peritoneal Dialysis," *The International Journal of Pediatric Nephrology*, 8 (1987), 199-202.

P. R. Uldall: "Temporary Vascular Access for Hemodialysis," *Dialysis Therapy*, 2nd edition, (1993), 5-10.

P. R. Uldall, M. DeBruyne, M. Besley, J. McMillian, M. Simons, R. Francoeur: "A New Vascular Access Catheter for Hemodialysis," *American Journal of Kidney Diseases*, 21 (1993), 270-277.

M. Agraharkar, S. Isaacson, D. Mendelssohn, J. Muralidharan, S. Mustata, G. Zevallos, M. Besley, P. R. Uldall: "Percutaneously Inserted Silastic Jugular Hemodialysis Catheters Seldom Cause Jugular Vein Thrombosis," *American Society for Artificial Internal Organs, Inc.*, 41 (1995), 169-172.

## ACKNOWLEDGEMENT

M. Cohen, M.D., Department of Pediatric Nephrology, Phoenix Children's Hospital, Phoenix, Arizona.

## VIDEOTAPES

**C-VC-3 COOK SUBCLAVIAN DOUBLE LUMEN HEMODIALYSIS CATHETER**

AUTHOR: H. Silberman, M.D., Department of Surgery, Los Angeles County U.S.C. Medical Center, Los Angeles, California. (Duration: 19 min.)

**C-VC-7 PEEL-AWAY® SHEATH TECHNIQUE FOR PLACING PERITONEAL DIALYSIS CATHETERS**

AUTHOR: S. Updike, M.D., Nephrologist, Nephrology Section, University of Wisconsin Medical Center, Madison, Wisconsin. (Duration: 17 min.)

**C-VC-9 SELDINGER TECHNIQUE FOR PLACEMENT OF THE TENCKHOFF CATHETER**

AUTHOR: A. R. Zappacosta, M.D., Chief, Section of Nephrology, Medical Director, Dialysis, The Bryn Mawr Hospital, Bryn Mawr, Pennsylvania. (Duration: 10 min.)

**C-VC-36 ULDALL DOUBLE LUMEN HEMODIALYSIS CATHETER TRAY**

AUTHOR: P. R. Uldall, M.D., Division of Nephrology, Wellesley Hospital, Toronto, Ontario. (Duration: 31 min.)

# COOK®

Interventional Critical Care Products

[www.cookgroup.com](http://www.cookgroup.com)

**COOK CRITICAL CARE**

P.O. Box 489, Bloomington, IN 47402-0489 U.S.A.  
Phone: 812 339-2235, Toll Free: 800 457-4500, Toll Free FAX: 800 554-8335

**COOK (CANADA) INC.**

111 Sandiford Drive, Stouffville, Ontario, L4A 7X5 CANADA  
Phone: 905 640-7110, Toll Free: 800 668-0300

**WILLIAM A. COOK AUSTRALIA PTY. LTD.**

Brisbane Technology Park, 12 Electronics Street, Eight Mile Plains  
Brisbane, QLD 4113 AUSTRALIA, Phone: +61 7 38 41 11 88

**WILLIAM COOK EUROPE ApS**

Sandet 6, DK-4632, Bjaeverskov, DENMARK, Phone: +45 56 86 86 86

© COPYRIGHT COOK INCORPORATED 2002

C-DB302